S.A.S Band Tracheostomy Tube Holder

Ref: A-Z 40-01

Instructions For Use










Step 1:


Thread the velcro tabs of both sections of the band through the eyelets in the tracheostomy tube and back onto the pink surface of the foam band and velcro down. (see fig. 1)

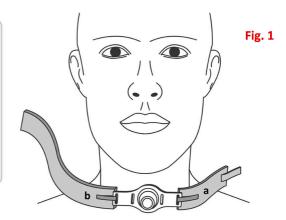

Step 2:

Bring longer piece (b) of foam band behind the patient's neck and underneath section (a). Adjust tension in the band and velcro down. Trim off any excess foam from section (b). (see fig. 2)

* NB. Always check the tracheostomy tube manufacturer's instructions before deploying the holder.

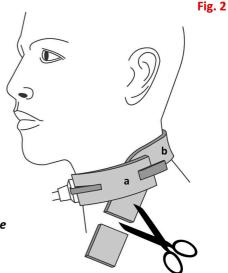

www.aztecmedical.co.uk

tel: 01303 487 891 | email: sales@aztecmedical.co.uk | fax: 01303 760 161

Version 3 July 2015

_. _